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ENGLISH

### Woundsolution

An alcohol free solution for cleansing, decontaminating, moisturizing and promoting healing of acute and chronic wounds.

#### Characteristics

- › For the pain-free physical wound cleansing of acute and chronic wounds
- › Effective decontamination of coated and contaminated wounds (e.g. MRSA)
- › Destruction of biofilm
- › Solves wound crusts and adherent dressing material
- › Prevents infections
- › Enhances wound healing
- › Overcomes the chronic-stagnative wound healing phase
- › Optimizes the environment on the wound surface (pH-level)
- › Reduces pain
- › Cooling effect
- › Use can be repeated several times a day
- › Can be used in combination with customary products/dressings for wound care

Acute wounds are often contaminated with dirt, debris or microorganism. In these cases the thorough and gentle wound cleansing is important for a fast wound healing. Chronic wounds are often coated with wound exudate, necrotic tissue or biofilms. These coatings delay the wound healing and therefore have to be removed. For both types of wounds a gentle and thorough wound cleansing and longlasting wound moisturizing with Allrinse® Woundsolution is indicated. Allrinse® Woundsolution offers an excellent cleansing activity on the wound surface and the surrounding skin.

#### Indications

- › For the cleansing and moisturizing of acute and chronic wounds
- › For the removal of adherent dressing material or crusts during dressing changes
- › For decontamination

#### Directions for use

Wound cleansing:

Cleanse wound with Allrinse® Woundsolution to loosen coatings and/or biofilm. Allrinse® Woundsolution can be applied undiluted directly out of the bottle into the wound. Bottle of 350 ml is intended for wound cleansing, spray of 75 ml is for spraying. Subsequently Allrinse® Woundsolution should be removed by swabbing with customary sterile gauze compress (mechanical cleaning). Rinsing afterwards is not necessary.

Moisturizing with dressings:

Soak customary dressings (e.g. compress, gauze) with Allrinse® Woundsolution and apply it to the wound for ca. 10–15 min. After the soaking time wounds should be cleaned mechanically with customary sterile gauze compress.

#### Contraindications

Do not use with known allergy or suspicion of allergic reactions to one of the ingredients. Do not use in combination with anionic surfactant. Do not moisten silver-containing dressings with Allrinse® Woundsolution. Avoid contact with cartilage-tissue. In the case of big, heavily wetting or infected wounds, contact a doctor, because in this case a special treatment is necessary. Contact a doctor for burns of degree 2b or higher. Application for pregnant or breastfeeding women, babies and toddlers/young children should only be carried out with mandatory indication after consultation of a doctor.

#### Tissue tolerance and biocompatibility

Allrinse® Woundsolution is non-cytotoxic, non-irritating and non-sensitizing. Allrinse® Woundsolution does not contain skin-irritating ingredients.

#### General safety instructions

Consider temperature limit. Do not use damaged packages. Only use faultless and undamaged packages. Protect from sunlight. Keep in a dry place. Consider instructions for use. Do not use after expiration date. For external use only. Do not swallow. Not for infusion or injection. Mind hygienic use. Dispose the tube immediately after contact of the tube opening with the wound or in the case of otherwise contamination. Keep out of the reach of children. Close after use to avoid contamination.

#### Shelf life/Expiration date

3 years after manufacturing

#### Time limit to use

3 months after opening

#### Packaging sizes

350 ml, 75 ml

 LOT-Number /  Expiration date  
 printed onto bottle

#### Ingredients (INCI)

Aqua purificata, Cocamidopropyl Betaine, Polyaminopropyl Biguanide (PHMB), Trace Elements (Zinc, Iron)